

How to Collect Oral Fluid

Quantisal Collection Device - For Drugs of Abuse Clients

If the donor has gum, ice, candy, etc. in their mouth, ask them to remove the object before beginning the collection process. The donor must abstain from food and drink for 10 minutes prior to oral fluid collection. Carefully instruct the donor to perform the following steps in order to properly conduct their own sample collection.

1




Instruct the donor to remove the contents of the Quantisal package

2




Without touching the pad, remove the collection device from its package by the plastic handle.

3

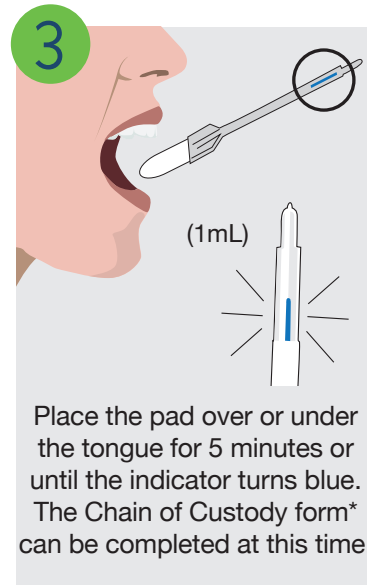


Place the pad over or under the tongue for 5 minutes or until the indicator turns blue. The Chain of Custody form* can be completed at this time.

4




Pop the cap off of the collection tube

5

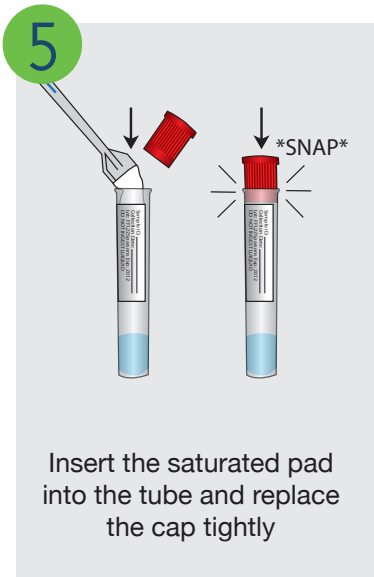


Insert the saturated pad into the tube and replace the cap tightly

6

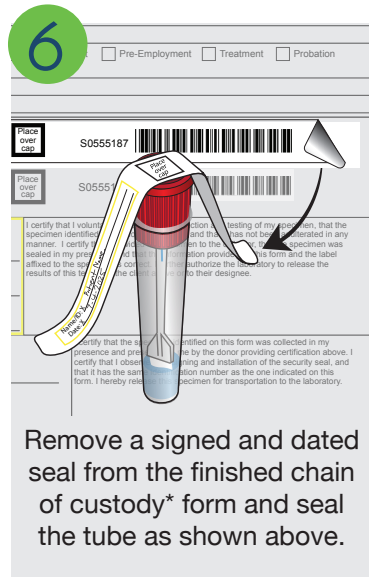

Remove a signed and dated seal from the finished chain of custody* form and seal the tube as shown above.

7

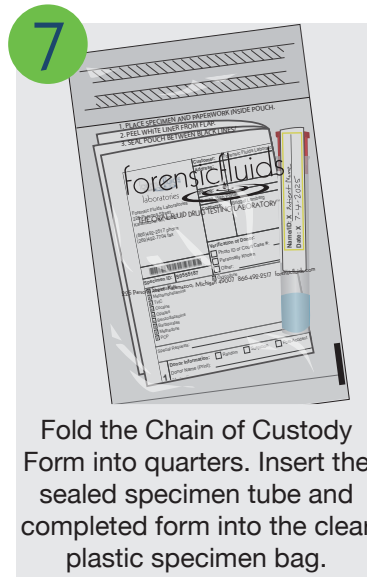

Fold the Chain of Custody Form into quarters. Insert the sealed specimen tube and completed form into the clear plastic specimen bag.

8



Seal the specimen bag and place it into the prepaid shipping bag. Send the shipping bag to Forensic Fluids for testing.

*See reverse side for chain of custody form instructions

The above instructions are intended for reference purposes only.

For questions or for full training, please call Forensic Fluids Laboratories at 866.492.2517



The Oral Fluid Drug Testing Specialists
866.492.2517 | forensicfluids.com
225 Parsons Street Kalamazoo, MI

